

CanaryBox® Quick Guide

Settings

Parameters

Select parameters desired.


Restore Defaults

Sets parameter thresholds back to defaults.

Brightness

Set display brightness.

Restore Factory Defaults - Yes/No: On power up, select the option to restore the CanaryBox® default settings (including parameter threshold values) prior to use. This is recommended prior to each case to fine tune the vitals parameter thresholds precisely to your patient. If the default settings are not restored, CanaryBox® will remember the last settings used.

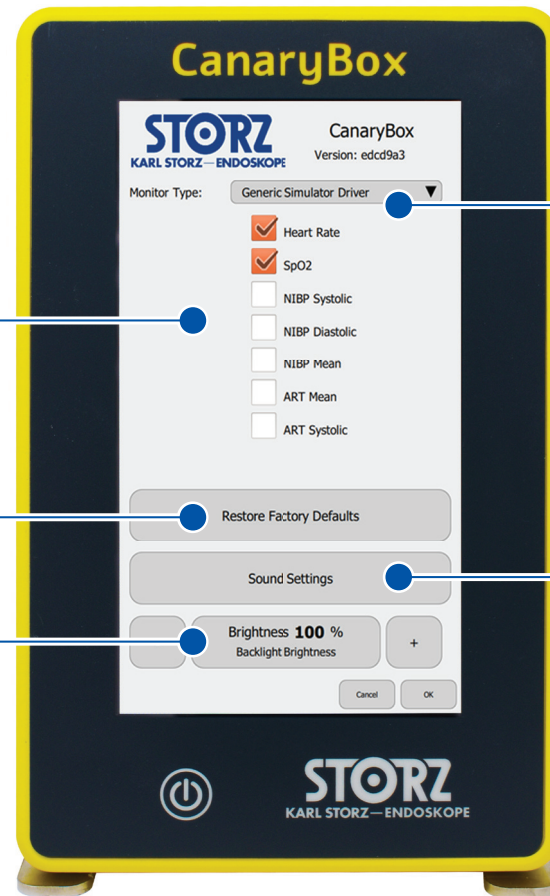

Monitor Type

Select manufacturer/model.

Sound Settings

Customize threshold actions on music volume.

Customizations

For use during cardiopulmonary bypass, add mean invasive blood pressure and deselect heart rate and SpO2.

If the pulse oximeter is on the same arm as the blood pressure cuff, lengthen the delay to avoid nuisance triggers.


NIBP can be deselected if there is excessive interference.

Heart rate is obtained first from the ECG. If the ECG is unavailable (e.g. electrocautery or a lead disconnection), the heart rate is obtained from the pulse oximeter.

IMPORTANT NOTE: This material is not a substitute for the Instruction Manual (IFU) for this product. Please always refer to the product Instruction Manual for detailed instructions.


CanaryBox® Quick Guide

Main Screen

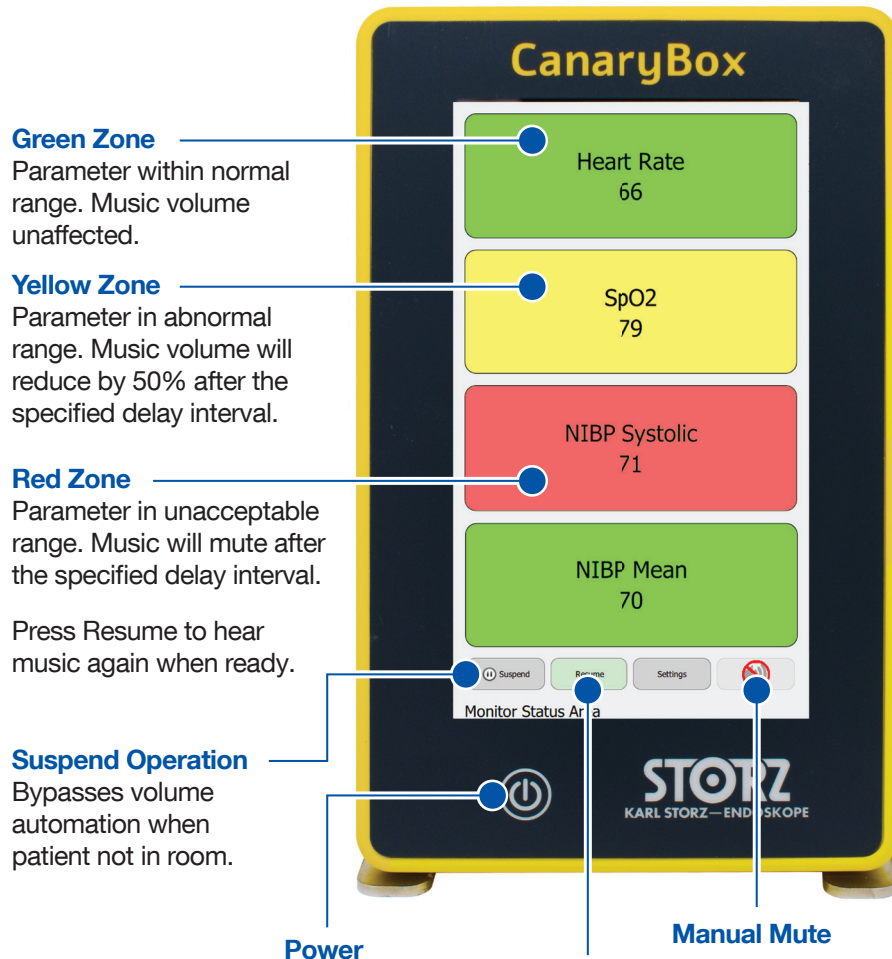

Tap any parameter to adjust the thresholds or delay interval for that parameter.

Connections

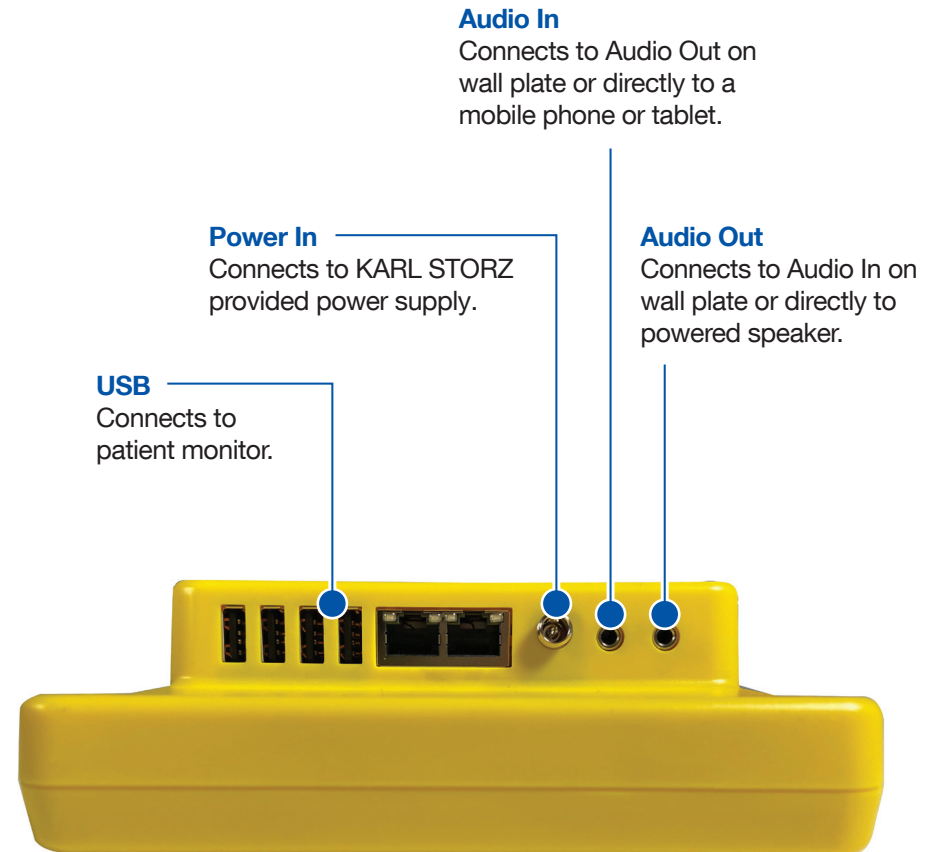

IMPORTANT NOTE: This material is not a substitute for the Instruction Manual (IFU) for this product. Please always refer to the product Instruction Manual for detailed instructions.