

# Clinical Efficacy and Efficiency of the Resolution 360™ Clip

## technique spotlight



**Jonh Pineda, MD**

Lewis Gale Medical Hospital  
Salem, Virginia, USA


### Patient History

A 68-year-old woman with a history of constipation and intermittent diarrhea underwent surveillance colonoscopy. During the colonoscopy, a large, laterally spreading flat polyp consistent with a sessile serrated adenoma was seen in the cecum.

### Procedure

The polyp was lifted and demarcated. The lesion was removed by endoscopic mucosal resection (EMR) technique, piecemeal fashion with a Captivator™ Single-Use Stiff Snare. The edges of the lesion were ablated with soft coagulation using the tip of the snare. Visible vessels and a bleeding spot were ablated with a coagulation grasper (**Figure 1**).

The 2.9 cm defect was closed with placement of twelve Resolution 360 Clips (**Figures 2 through 5**).

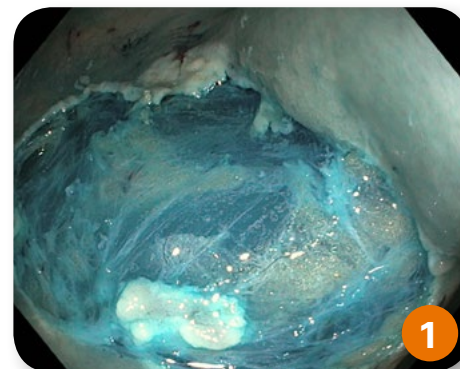

# Clinical Efficacy and Efficiency of the Resolution 360™ Clip

## technique spotlight

### Post Procedure

The patient tolerated the procedure well. The polyp was a sessile serrated adenoma without high grade dysplasia. A surveillance colonoscopy will be done in six months.

### Discussion

This case demonstrated the ability to effectively close a large post EMR defect using the new Resolution 360 Clip. The one-to-one physician control to rotate the clip afforded fast and easy placement of the clips in the exact alignment needed to close the defect in this case. The jaw strength of the clip allowed for the edges of the lesion to be brought together reliably. There were no clip misplacements contributing to a shorter procedure time and cost effectiveness.



All trademarks are the property of their respective owners.

Results from case studies are not necessarily predictive of results in other cases. Results in other cases may vary.

**CAUTION: Federal (USA) law restricts these devices to sale by or on the order of a physician.**

**CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Information for use only in countries with applicable health authority registrations. Material not intended for use in France.**

Boston Scientific Corporation  
300 Boston Scientific Way  
Marlborough, MA 01752-1234  
[www.bostonscientific.com](http://www.bostonscientific.com)

©2017 Boston Scientific Corporation  
or its affiliates. All rights reserved.

ENDO-407412-AB May 2017