


Directions for use

BD PureWick™ Urine Collection System and BD PureWick™ Male External Catheter


PureWick™ Urine Collection System setup

1. Plug power cord into device outlet and into the A/C power outlet. Note: Make sure the power switch is turned off.
2. Place collection canister in the base and press down firmly on the lid of the device making sure lid is sealed.
3. Optional: Slip privacy cover onto the canister prior to placing canister into base.
4. Attach pump tubing (short tube) to the PureWick™ Urine Collection System connector port and the connector port on the collection canister lid.
5. Attach the elbow connector to the collector tubing (long tube) and connect elbow connector to the connector port on collection canister lid.
6. Connect other end of collector tubing (long tube) to a PureWick™ External Catheter.

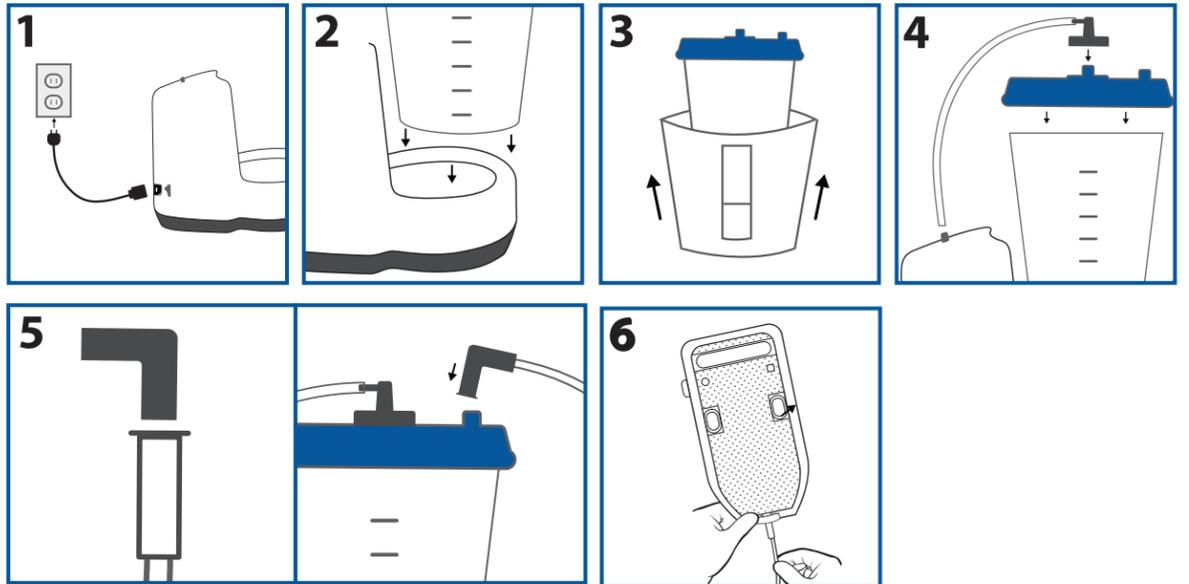


PureWick™ Male External Catheter placement

7. **A.** Trim or clip the pubic hair and check the skin for irritation or breakdown.
B. Clean the penis and skin around the base of penis with soap and water. Dry skin before positioning the device.
8. **A.** Position device, aligning drainage fitting toward feet of user. Slide the opening of the device over the penis. Do not place the scrotum inside the device.
B. Remove the top adhesive peel-off and press the device against the skin. Repeat for the bottom adhesive peel-off.

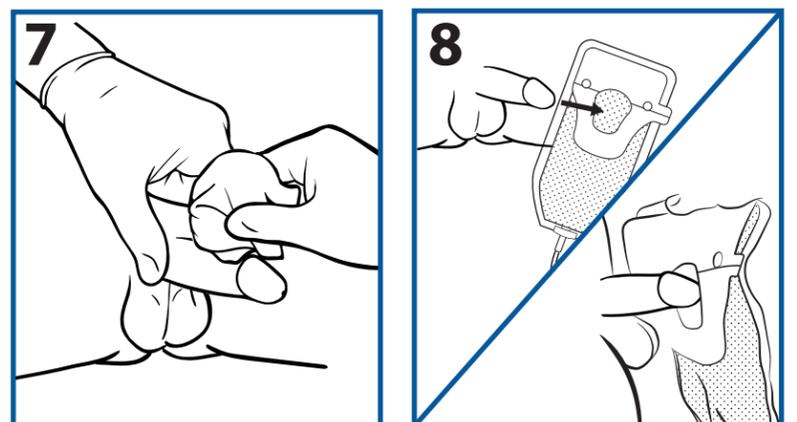

PureWick™ Male External Catheter removal and maintenance

9. Gently lift the top edge of the adhesive base off the skin. In a slow, downward peeling motion, remove the device. If necessary, use a warm, wet cloth to help loosen adhesive.
10. Replace at least every 24 hours or if soiled with feces, blood, or semen. Assess skin for irritation and clean and dry the skin prior to placement of a new PureWick™ Male External Catheter.

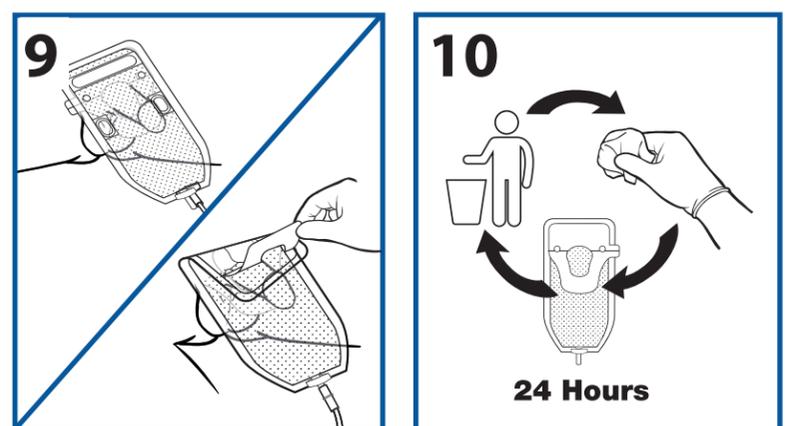

See reverse side for indications, contraindications, safety and warnings.

C. R. Bard, Inc., Covington, GA 30014, U.S.
844.823.5433

bd.com

BD, the BD Logo and PureWick are trademarks of Becton, Dickinson and Company.
© 2023 BD and its subsidiaries. All rights reserved. 2311-30 BD-110051



Indications: The PureWick™ Urine Collection System is to be used with the PureWick™ External Catheters which are intended for non-invasive urine output management, such as urinary incontinence.

Contraindications: Do not use the PureWick™ Urine Collection System with PureWick™ Male External Catheters on individuals with urinary retention.

Warnings: Do not use on irritated skin. Do not cover fresh surgical wounds with the device. Do not use the PureWick™ Male External Catheter with any material or in any position that blocks the vents on the front of the device or does not allow for airflow through the device. To avoid potential skin injury, never pull the PureWick™ Male External Catheter direct away from the user. Always peel in the direction from head to foot. Stop use if an allergic reaction occurs. After use, this product may be a potential biohazard. Dispose of in accordance with applicable local, state and federal laws and regulations.

Precautions: Not recommended for patients who: Are agitated, combative, or uncooperative and might remove the PureWick™ Male External Catheter. Have frequent episodes of bowel incontinence without a fecal management system in place. Have skin irritation or breakdown at the site. Do not use barrier creams, lotions, or ointments on the penis or pubic skin around the base of the penis when using the device. These may impede suction or weaken adhesive. Proceed with caution in users who have had recent surgery of the external male anatomy. Always check skin for irritation or breakdown and clean and dry the skin prior to placement of a new device.

Safety and Warnings: Always unplug PureWick™ Urine Collection System before cleaning or when not in use. Do not immerse the PureWick™ Urine Collection System in water. As with most electrical devices, electrical parts in this system are electrically live even when the power is off. To reduce the risk of electric shock, if the PureWick™ Urine Collection System falls into water, unplug immediately. Do not reach into the water to retrieve it. Contains small parts that may cause choking. Keep out of reach of children. Keep cords and tubing out of the reach of children to avoid the risk of strangulation. Discontinue use if an allergic reaction occurs. Not recommended for users who are experiencing skin irritation or skin breakdown in device contact areas. WARNING: This device should not be used in oxygen rich environments or in conjunction with flammable anesthetics. If the PureWick™ Urine Collection System is dropped or tipped over spilling urine, unplug the unit and carefully inspect for loose or damaged parts before resuming use. Contact Customer Support if any damage is observed. The pump tubing connecting the PureWick™ Urine Collection System to the collection canister may experience light condensation inside the tube. This is not unusual and does not affect function. However, if urine or water has streamed into the pump, discontinue use and contact Customer Support. Although the canister can hold up to 2000cc (mL), the urine should be emptied regularly from the collection canister before volume reaches 1800cc (mL). Failure to empty canister before urine overflow may cause damage to the PureWick™ Urine Collection System and is not covered under the warranty. It is important that the port connections be connected correctly for proper operation of the PureWick™ Urine Collection System. Use of this equipment next to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally. Portable radio frequency (RF) communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 inches (30cm) to any part of the PureWick™ Urine Collection System, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result. Use only PureWick™ Urine Collection System accessories with this device. Incompatible parts or accessories can result in degraded performance and will void the warranty. Do not use accessories past expiration date indicated on package labeling. Use only the PureWick™ Urine Collection System A/C power cord with the device. Use of an alternate consumer style A/C power adapter or an extension cord may cause damage to device and electrical shock or injury and will void the warranty. Do not place PureWick™ Urine Collection System or its cord across walkways creating a tripping hazard. For PW200: The battery cells used in this device may present a fire or chemical hazard. To minimize the risk of damaging the battery inside the PureWick™ Urine Collection System, do not use the device outside the indicated operating temperature range of 41°F - 104°F (5°C - 40°C). The battery is not intended to be replaced; replacement could result in a hazard. To reduce the risk of electrical shock or injury, do not disassemble this unit. No modification of this equipment is allowed



PureWick™ System for men