

BBS Revolution™ Bladder Scanner

Cleaning instructions:

- After each use, wipe the scanner with a soft cloth or towel to remove excess gel. A moist cleaning cloth may also be used to clean the unit.
- Disinfect the unit with bleach wipes, alcohol wipes or any hospital disinfectant such as CaviCide®, CaviWipes®, Cidex®, Cidex 7®, or Sporocidin®.
- After cleaning, inspect the scanner for cracks or damage.
- Caution: Do not use harsh, corrosive chemicals (e.g. hydrochloric acid) or Cidex Plus® to perform maintenance.



The BBS Revolution™ bladder scanner device projects ultrasound energy through the lower abdomen of the patient to obtain an image of the bladder, which is used to calculate bladder volume non-invasively. Device is for external use only. Do not use device on patients with open or damaged skin, or wounds in the suprapubic region. Scanner used without ultrasound gel may result in a poor or inaccurate reading. Users should consult with a healthcare professional for the interpretation of the results. Please consult product label, inserts, and instructions for use for any indications, contraindications, hazards, warnings, cautions, and directions for use.

Distributed by:
C. R. Bard, Inc., Covington, GA 30014, U.S.
844.823.5433

bd.com

Manufactured by:
dBMEDx Inc., 7761 Shaffer Parkway, Suite 140
Littleton, CO 80127 U.S.

dBMEDx.com