




Kangaroo™

KANGAROO OMNI™ Enteral Feeding Pump

Quick Reference Guide

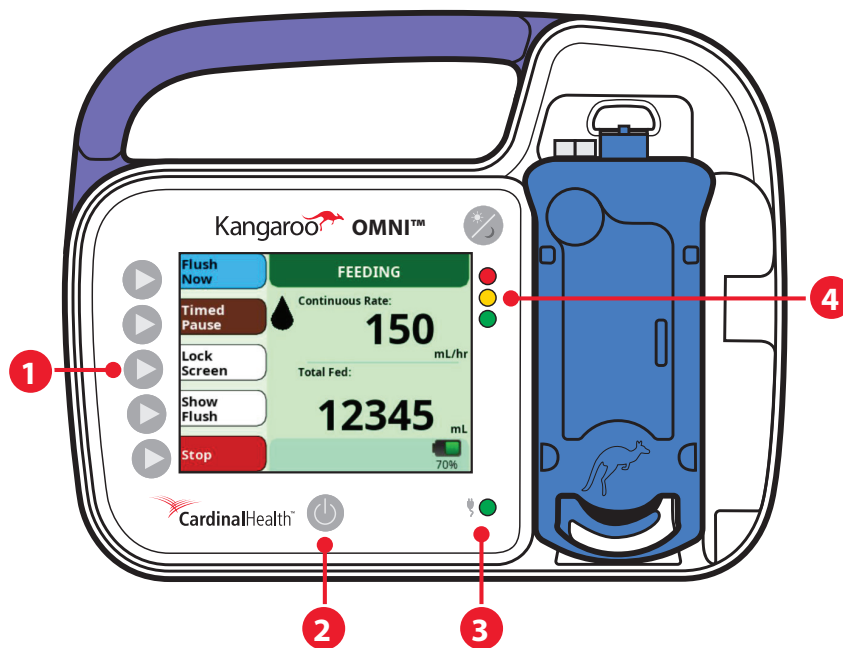

1. Selection Buttons

Press a key to select the option that appears next to it on the screen.

2. Power Button

Press once to power on.
Press and hold to power off.

3. Power Source

Shines a green light when the pump is connected to AC power.


4. Pump Status

- = Warning Alarm
- = Notice or Caution Alarm
- = Standby or Feeding


See reverse for more information.




Set Up the Pump

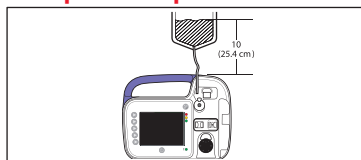


1. Hang the feeding bag or feeding container so the highest level of enteral fluids is 10 inches (25.4 cm) above the top of the pump.

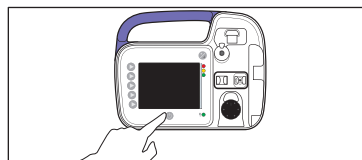

2. Press **Power** button.

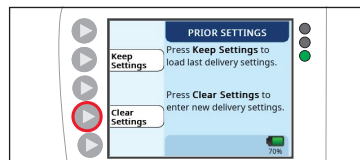


3. Select **Clear Settings** to enter new feed parameters. Select **Keep Settings** to keep old feed parameters.

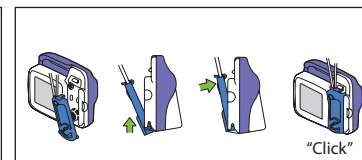


4. Load Set. Insert from bottom and snap into place.


Prime the Pump

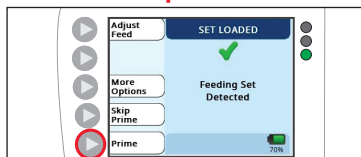


1. Select **Prime**. WARNING: Patient must not be connected to Kangaroo OMNI™ Enteral Feeding Pump when priming is performed.

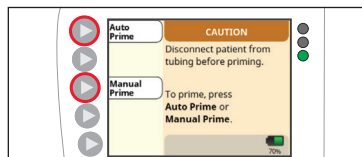

2. Select Priming Mode: either **Auto Prime** or **Manual Prime**.

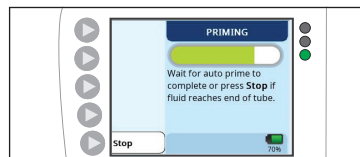


If you selected Auto Prime:
3. Wait for pump to finish priming.

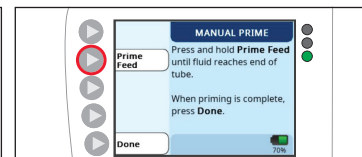


If you selected Manual Prime:
4. Press and hold **Prime Feed** until fluid reaches end of tubing.


Start a New Feed

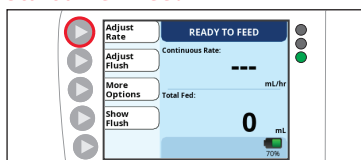

If you selected Clear Settings:
1. Select **Adjust Rate**.

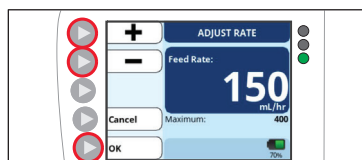


2. Press the + and - buttons to enter the rate, then select **OK**.

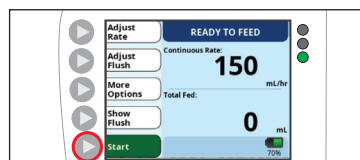


3. When ready to begin feeding, select **Start**.


Start a New Flush:

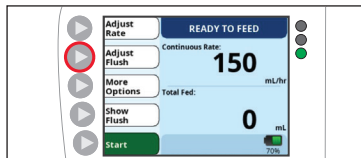


1. Select **Adjust Flush** button.

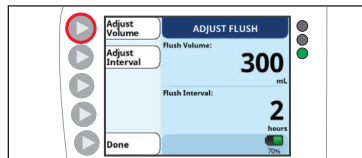


2. Press **Adjust Volume** button.

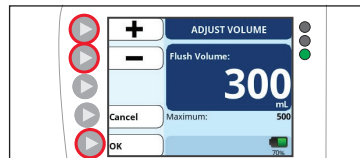


3. Press + and - buttons to enter the flush volume, then select **OK**.

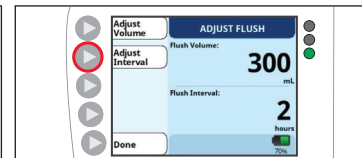

4. Press **Adjust Interval** button.

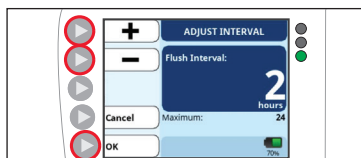


5. Press the + and - buttons to input the desired time interval, and then press **OK**.

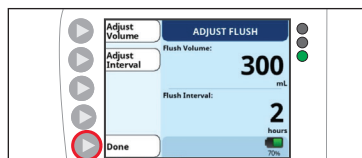


6. Confirm the flush volume and time interval, then press **Done** to return to the **Ready to Feed** screen.

Pause the Feed

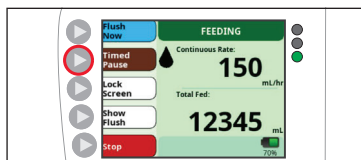


1. To suspend the feeding, select **Timed Pause**.

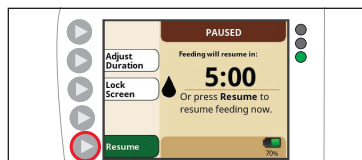

2. To resume feeding, select **Resume**. Pump will resume on its own after time expires.

Power Down the Pump

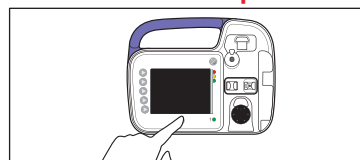

1. Press **Power** button.

For additional operations or information, see the Kangaroo OMNI™ Enteral Feeding Pump Instructions for Use.



mycardinalmsds.com



Follow instructions for use. Symbol appears blue on device.

MR Unsafe

Avoid temperature extremes

MR Unsafe – an item that is known to pose hazards in all MR environments. Visit the Kangaroo™ Learning Center at www.mykangaroolearning.com for more information.

Cardinal Health 200, LLC

3651 Birchwood Drive
Waukegan, IL 60085 USA
Rev. A 2024-07 • cardinalhealth.com • Lit. No. 2MS24-3028612

© 2024 Cardinal Health. All Rights Reserved. CARDINAL HEALTH, the Cardinal Health LOGO, KANGAROO, the Kangaroo LOGO and KANGAROO OMNI are trademarks of Cardinal Health and may be registered in the US and/or in other countries. Patent cardinalhealth.com/patents.