


Hysteroscopes from KARL STORZ

Diagnostic and Operative Solutions
for Hysteroscopy

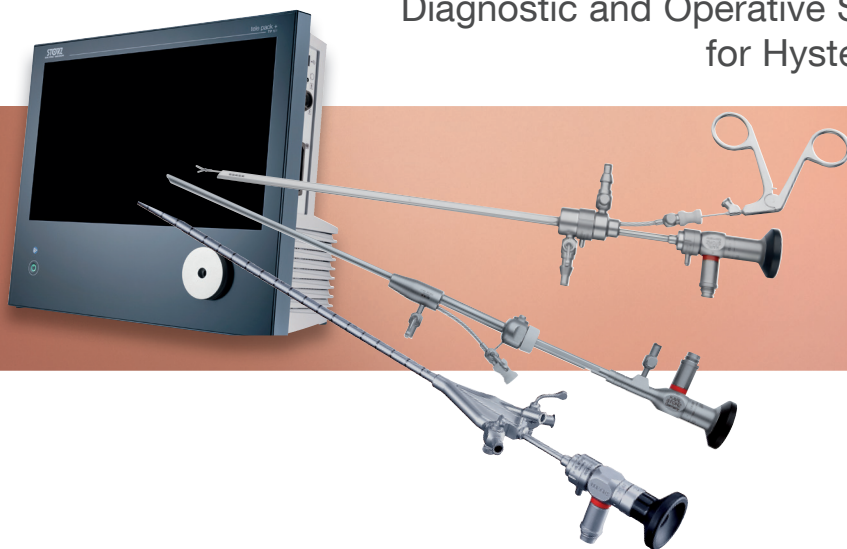

CAMPO TROPHYSCOPE®

Based on integrated 2 mm rod lens telescope

Special Features:

- Unique possibility of atraumatic insertion in the cervical canal without dilation and anesthesia due to very small outer diameter
- Innovative feature: Sheath with gliding mechanism to ensure guided dilation
 - Primary approach to uterine cavity with the first 5 cm being only 2.9 mm outer diameter
 - Intraoperative changeover from single-flow to continuous-flow and/or operating sheath
- TROPHYSCOPE® features greater stability thanks to integrated irrigation channel
- Working channel allows the use of semi-rigid 5 Fr. operating instruments and bipolar electrodes
- Brilliant image quality thanks to HOPKINS® rod lens system

Without working channel



26008BAC CAMPO **TROPHYSCOPE®**, HOPKINS® telescope 30°, size 2.9 mm, length 24 cm, with irrigation connector, for use with Continuous-Flow Operating Sheaths 26152DA and 26152DB

26152DA **Continuous-Flow Examination Sheath**, size 3.7 mm, length 18 cm, with suction adaptor, for use with CAMPO TROPHYSCOPE® 26008BAC

With working channel




26008BAC CAMPO **TROPHYSCOPE®**, HOPKINS® telescope 30°, size 2.9 mm, length 24 cm, with irrigation connector, for use with Continuous-Flow Operating Sheaths 26152DA and 26152DB

26152DB **Continuous-Flow Operating Sheath**, size 4.4 mm, length 16 cm, with channel for semi-rigid instruments 5 Fr., with 1 stopcock and 1 Luer-Lock adaptor, for use with CAMPO TROPHYSCOPE® 26008BAC

BETTOCCHI® Hysteroscope (4 mm)

Based on 2 mm rod lens telescope

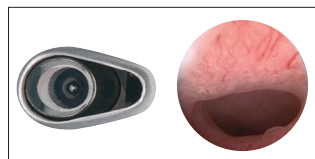


Special Features:

- Atraumatic insertion in the cervical canal due to oval sheath
- Very small outer diameter of only 4 mm
- Working channel allows the use of semi-rigid 5 Fr. operating instruments and bipolar electrodes
- Brilliant image quality thanks to HOPKINS® rod lens system



26008BA **HOPKINS® Forward-Oblique Telescope 30°**, diameter 2 mm, length 26 cm, **autoclavable**, fiber optic light transmission incorporated, color code: red


With working channel

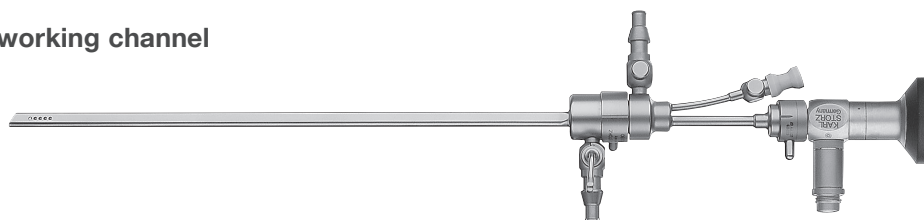

26152BIK **BETTOCCHI® Inner Sheath**, size 3.6 mm, with working channel for semi-rigid 5 Fr. operating instruments, for use with Outer Sheath 26152BOK including:
LUER-Lock Tube Connector

26152BOK **BETTOCCHI® Outer Sheath**, size 4.2 mm, working length 17.8 cm, for use with Inner Sheath 26152BIK including:
LUER-Lock Tube Connector

Without working channel

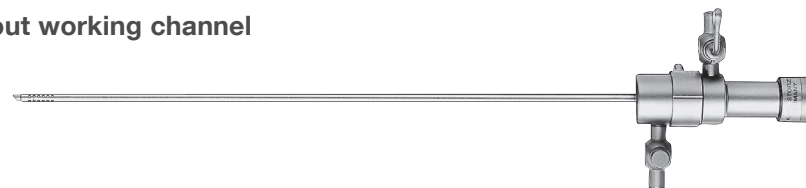


26161 RNK **Inner Sheath**, diameter 2.8 mm, with 1 stopcock, for use with Outer Sheath 26161RK including:
LUER-Lock Tube Connector

26161 RK **Continuous-Flow Outer Sheath**, diameter 3.6 mm, working length 20.7 cm, with 1 stopcock, for use with Inner Sheaths 26161RNK and 26162RNK including:
LUER-Lock Tube Connector

BETTOCCHI® Hysteroscope (5 mm)

Based on 2.9 mm rod lens telescope

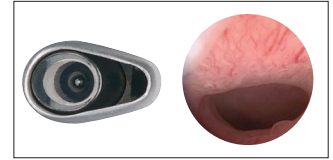


Special Features:

- Atraumatic insertion in the cervical canal due to oval sheath
- Working channel allows the use of semi-rigid 5 Fr. operating instruments and bipolar electrodes
- Brilliant image quality thanks to HOPKINS® rod lens system



26120BA **HOPKINS® Forward-Oblique Telescope 30°**, diameter 2.9 mm, length 30 cm, **autoclavable**, fiber optic light transmission incorporated, color code: red

With working channel

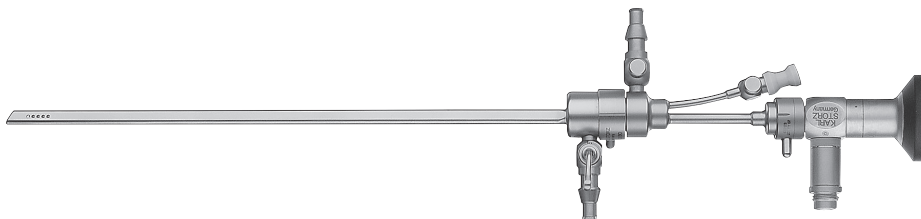

26153BIK **BETTOCCHI® Inner Sheath**, size 4.3 mm, with working channel for semi-rigid 5 Fr. operating instruments, for use with Outer Sheath 26153BOK including:
LUER-Lock Tube Connector


26153BOK **BETTOCCHI® Outer Sheath**, size 5 mm, working length 22.1 cm, for use with Inner Sheath 26153BIK including:
LUER-Lock Tube Connector

Without working channel



26161VBK **Inner Sheath**, diameter 3.8 mm, with 1 stopcock,
for use with Outer Sheath 26161VCK
including:
LUER-Lock Tube Connector

26161VCK **Outer Sheath**, diameter 4.5 mm, working length 24.8 cm, with 1 stopcock,
for use with Inner Sheath 26161VBK
including:
LUER-Lock Tube Connector

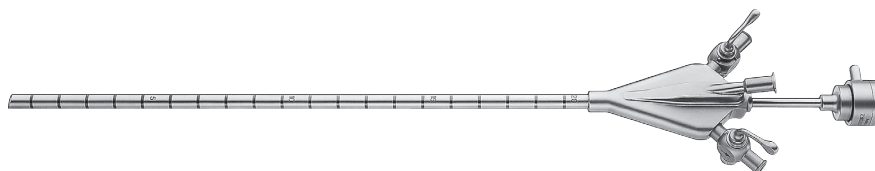

26153EA **High Flow Single Sheath**, size 5 mm, with channel for 5 Fr. operating
instruments, no separate inner and outer sheaths.
Can be used with a 12° or 30° telescope.

Extended Length BETTOCCHI® Hysteroscope

26120BAL **HOPKINS® Forward-Oblique Telescope 30°**, diameter 2.9 mm,
length 36 cm, **autoclavable**, fiber optic light transmission incorporated,
color code: red


26153BIL **BETTOCCHI® Inner Sheath**, size 4.3 mm, with channel
for 5 Fr. operating instruments, with 1 stopcock,
for use with Outer Sheath 26153BOL

26153BOL **BETTOCCHI® Outer Sheath**, size 5 mm, working length 28 cm,
with 1 stopcock, for use with Inner Sheath 26153BIL

The matching mechanical 5 Fr. instruments for the working channel bear the code number
26160XX.

Semi-rigid and Reusable 5 Fr. Operating Instruments

Available lengths: 34 cm or 40 cm (26160xx)

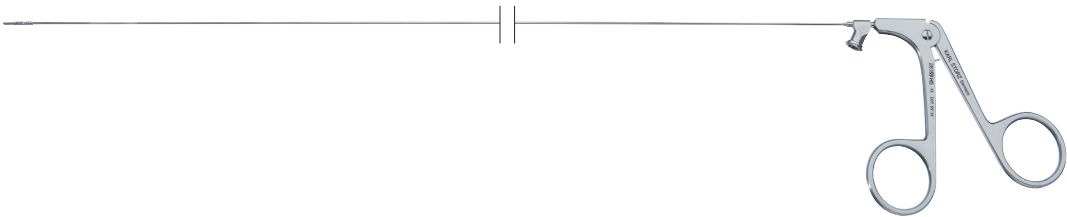


26159EHW **Scissors**, semi-rigid, blunt, single action jaws, 5 Fr., length 34 cm



26159SHW **Scissors**, semi-rigid, pointed, single action jaws, 5 Fr., length 34 cm



26159UHW **Biopsy and Grasping Forceps**, semi-rigid, double action jaws, 5 Fr., length 34 cm

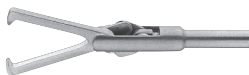

26159H **HESSELING Tenaculum Grasping Forceps**, semi-rigid, double action jaws, 5 Fr., length 34 cm



26159DHW **Punch**, semi-rigid, through-cutting, single action jaws, 5 Fr., length 34 cm



26159BHW **Biopsy Spoon Forceps**, semi-rigid, double action jaws, 5 Fr., length 34 cm



26159DS **DI SPIEZIO SARDO Grasping Forceps**, semi-rigid, double action jaws, 5 Fr., length 34 cm



26159HS **HESSELING and DI SPIEZIO SARDO Tenaculum Grasping Forceps with Spike**, semi-rigid, double action jaws, 5 Fr., length 34 cm

Reposable Bipolar 5 Fr. Electrodes

Length 42 cm



26159BE **Bipolar Dissection Electrode**, semi-rigid,
5 Fr., length 42 cm



26158BE **Bipolar Vaporization Electrode**, semi-rigid, 5 Fr.,
needle electrode angled 90°, length 42 cm



26159GC **GORDTS/CAMPO Bipolar Ball Electrode**,
semi-rigid, 5 Fr., length 42 cm




26176LV **Bipolar High Frequency Cord**, length 300 cm

Flexible Hysteroscope

Special Features:

- Insertion in the cervical canal without dilation due to very small diameter of 3.5 mm
- Easy insertion in the cervical canal thanks to reinforced sheath
- Lockable distal deflection mechanism for better visualization of tubal ostia

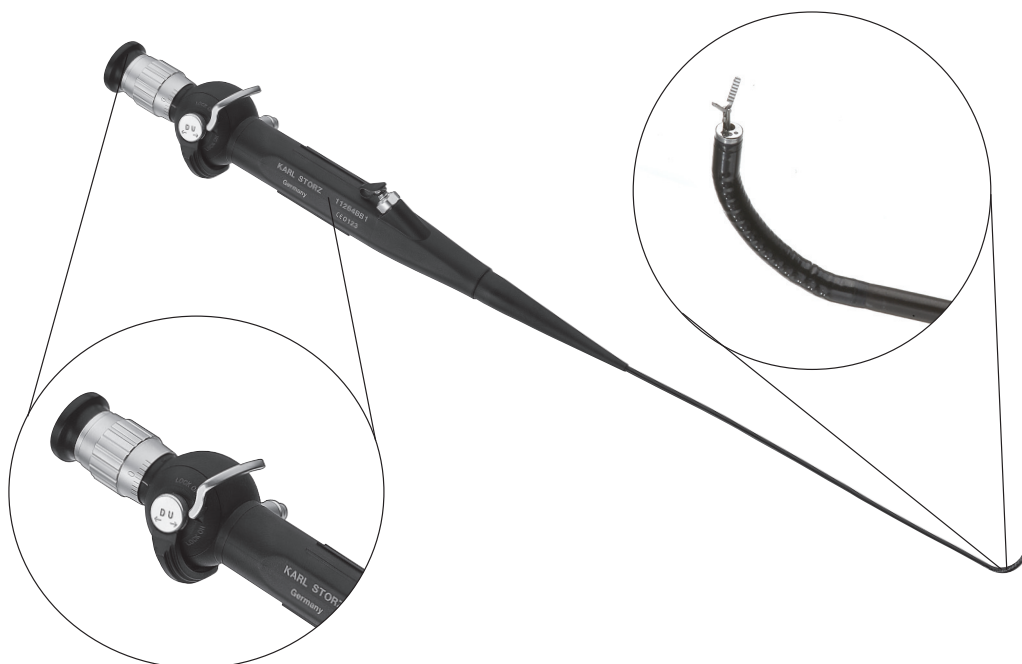

11264BBUK1 **Hystero-Fiberscope**, with lockable deflection mechanism,

Deflection of distal tip:	110°/110°
Direction of view:	0°
Working channel diameter:	1.48 mm
Outer diameter:	3.5 mm/3.6 mm
Working length:	240 mm

The following accessories are included in delivery:

Leakage Tester – 13242XL

Cleaning Brush – 27652-05, 11276CL2/10

Pressure Cap – 11025E

Double Stopcock – 6927691

KARL STORZ TELE PACK+

The TELE PACK+ is a portable video system that combines all the essential components (light source, camera control unit, and display) needed to enable hysteroscopic diagnosis and treatment as well as image capture and data management, making this solution ideal for all sites of care.

The TELE PACK+ provides superior visualization of hysteroscopic findings on the brilliant 18.5" full HD LED touchscreen display. Store images, videos, audio recordings and patient information in a 50GB internal hard drive, then transfer to your preferred portable media via USB.

In combination with KARL STORZ hysteroscopes, this unit provides a high-quality, multifunctional and compact system for outpatient hysteroscopies.



- | | |
|-------|--|
| TP101 | TELE PACK+ , endoscopic video unit for use with all KARL STORZ HX, high definition camera heads, video endoscopes and single-use scopes, including LED light source similar to XENON technology, with integrated digital Image Processing Module, 18.5" touch screen monitor including:
400A Power Cord , length 8' |
| TH110 | IMAGE1 HX Camera Head , 1080P HD for use with IMAGE1 S™ X-LINK, programmable camera head buttons |

Recommended Set-ups for “Office Hysteroscopy”

Outsourcing hysteroscopic procedures makes it possible to free up OR capacity for major surgery. To accommodate the trend towards “office hysteroscopy”, we have combined the following compact set-up solutions for outpatient hysteroscopy for your convenience.

The rolling mobile stand makes it possible to adjust the height of the TELE PACK+ unit and the equipment rails as required. Equipped with the HOPKINS® rod lens system, the office hysteroscopes from KARL STORZ in conjunction with the latest suction and irrigation pumps provide optimal visualization with brilliant image quality and a minimally invasive approach.

“TROPHY” Set-up

9700TPX	Rolling Stand for TELE PACK+
TP101	TELE PACK+
TH110	IMAGE1 HX Camera Head
UP210	ENDOMAT® SELECT , suction or irrigation pump
UP602	HYSTEROSCOPY Software , license, for use with ENDOMAT® SELECT UP210
UP610	Advanced Software , license, for use with ENDOMAT® SELECT UP210
26008BAC	CAMPO TROPHYSCOPE® , HOPKINS® telescope 30°, size 2.9 mm, length 24 cm
26152DA	Continuous-Flow Examination Sheath , size 3.7 mm, length 18 cm
26152DB	Continuous-Flow Operating Sheath , size 4.4 mm, length 16 cm
26159UHW	Biopsy and Grasping Forceps , semi-rigid, double action jaws, 5 Fr., length 34 cm
26159SHW	Scissors , semi-rigid, pointed, single action jaws, 5 Fr., length 34 cm

Accessories:

031523-10-GYN	ENDOMAT® SELECT Tubing Set for Irrigation , sterile, for single use, package of 10
260814FM-5	Fluid Management Pack 5 packs/box each pack includes: • Under buttocks drape • Cysto irrigation set inflow w/ LUER lock • Outflow drainage collection bag



Equipped with various software modules, ENDOMAT® SELECT is a roller pump that can be used for multiple surgical disciplines. In the operating room as well as the office setting or the clinical outpatient department, ENDOMAT® SELECT enables pressure-controlled distention for both diagnostic and operative hysteroscopy.

“BETTOCCHI®” Set-up

9700TPX	Rolling Stand for TELE PACK+
TP101	TELE PACK+
TH110	IMAGE1 HX Camera Head
UP210	ENDOMAT® SELECT , suction or irrigation pump UP
UP602	HYSTEROSCOPY Software , license, for use with ENDOMAT® SELECT UP210
UP610	Advanced Software, license, for use with ENDOMAT® SELECT UP210
26153BIK	BETTOCCHI® Inner Sheath , size 4.3 mm, with working channel for semi-rigid 5 Fr. operating instruments
26153BOK	BETTOCCHI® Outer Sheath , size 5 mm
26120BA	HOPKINS® Forward-Oblique Telescope 30° , diameter 2.9 mm, length 30 cm
26159UHW	Biopsy and Grasping Forceps , semi-rigid, double action jaws, 5 Fr., length 34 cm
26159SHW	Scissors , pointed, single action jaws, semi-rigid, 5 Fr., length 34 cm


Accessories:


031523-10-GYN	ENDOMAT® SELECT Tubing Set for Irrigation , sterile, for single use, package of 10
260814FM-5	Fluid Management Pack 5 packs/box each pack includes: • Under buttocks drape • Cysto irrigation set inflow w/ LUER lock • Outflow drainage collection bag




Equipped with various software modules, ENDOMAT® SELECT is a roller pump that can be used for multiple surgical disciplines. In the operating room as well as the office setting or the clinical outpatient department, ENDOMAT® SELECT enables pressure-controlled distention for both diagnostic and operative hysteroscopy.

It is recommended to check the suitability of the product for the intended procedure prior to use.
Please note that the described products in this medium may not be available yet in all countries due to different regulatory requirements.

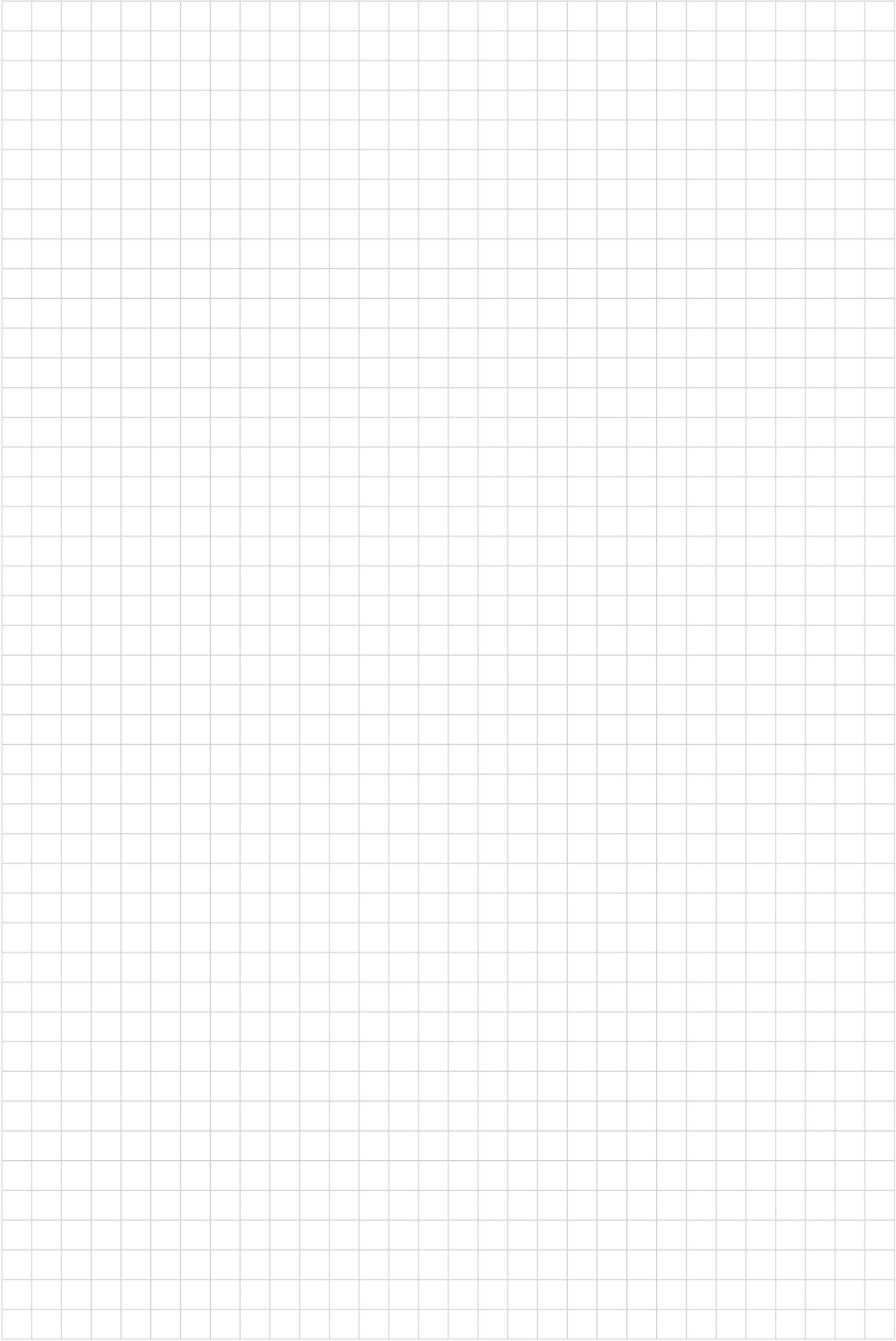

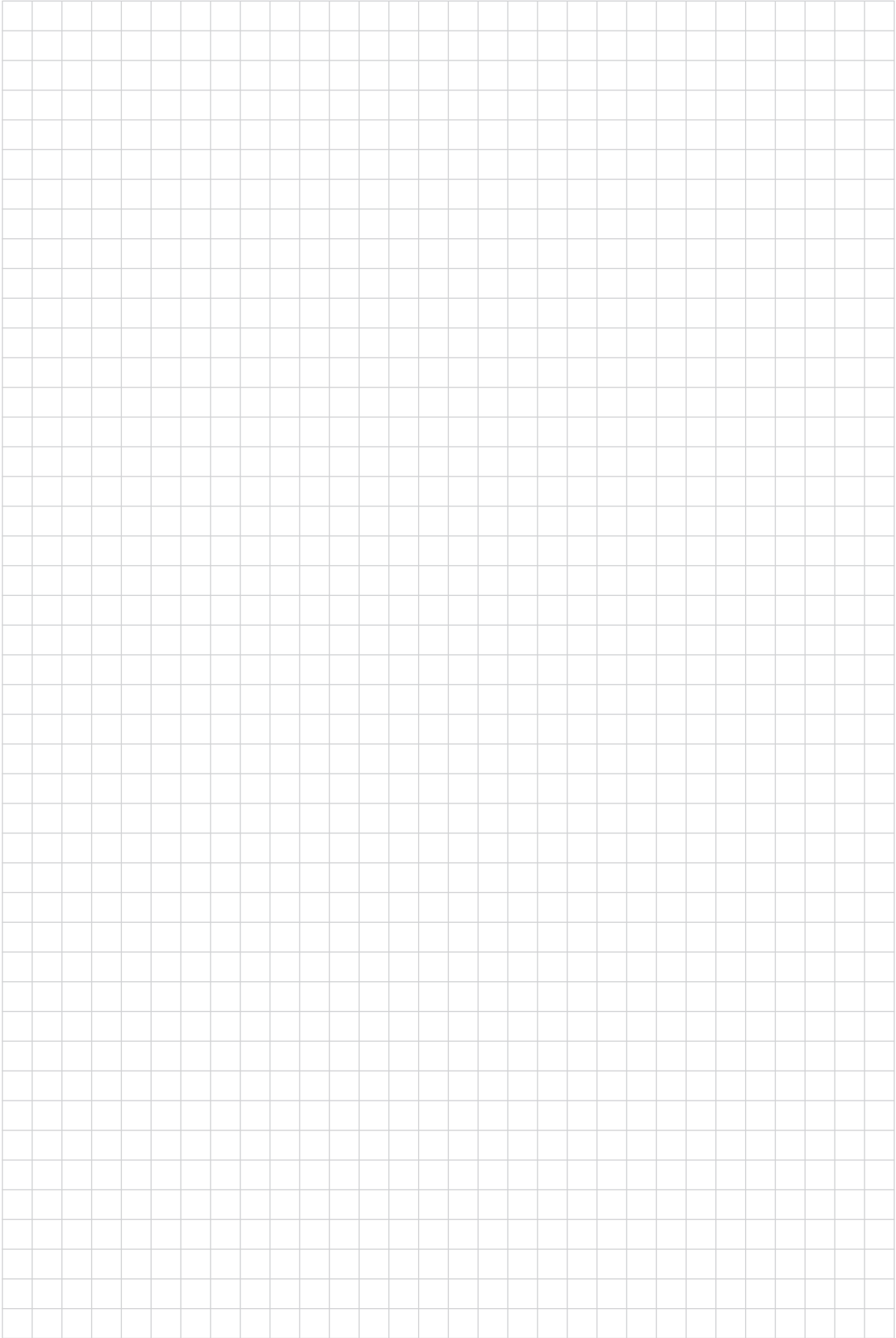

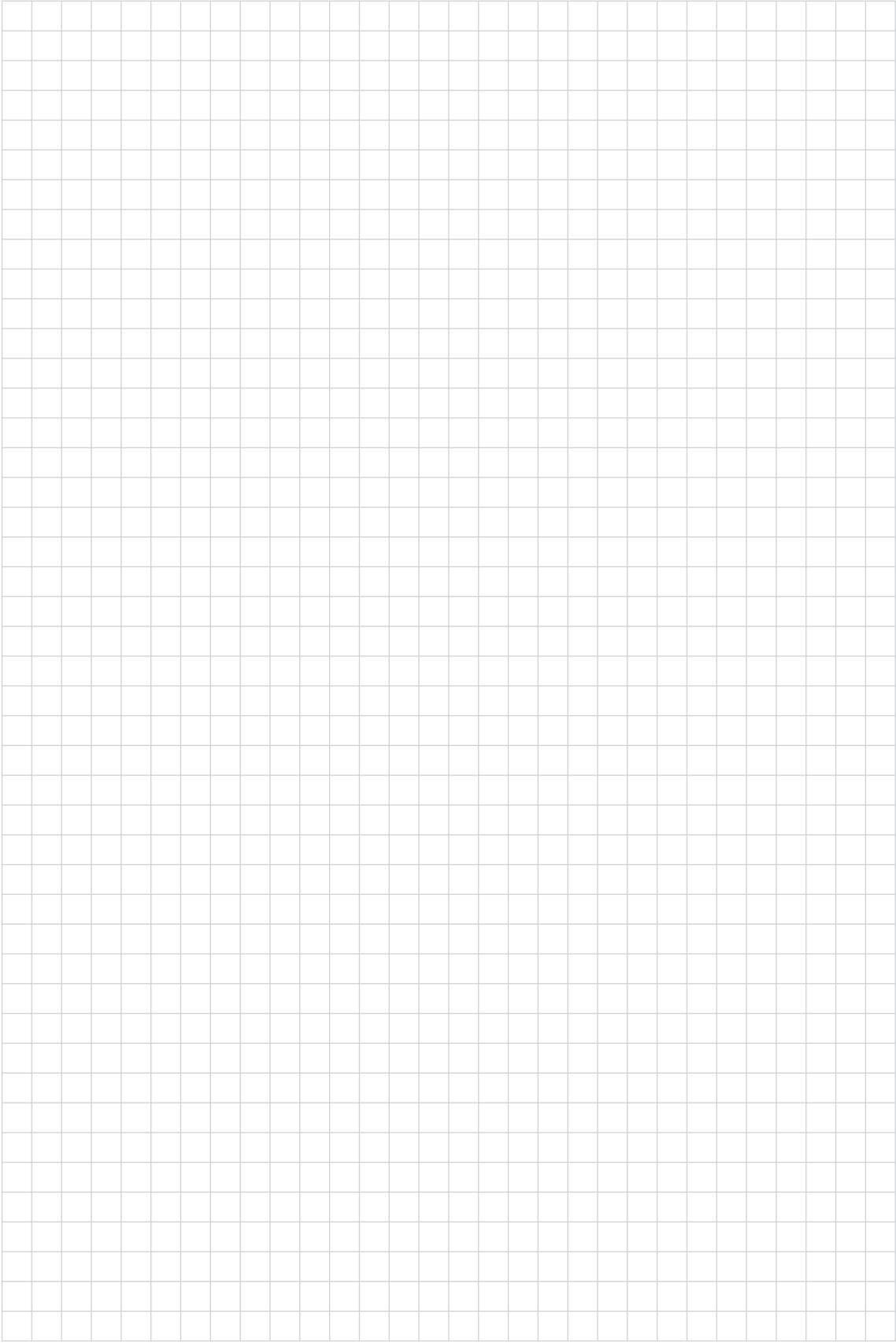

More than
75
Years

*Shaping the Future
of Endoscopy with you*

KARL STORZ Endoscopy-America, Inc.
2151 East Grand Avenue
El Segundo, CA 90245-5017, USA
Phone: +1 424 218-8100
Phone toll free: 800 421-0837 (US only)
Fax: +1 424 218-8525
Fax toll free: 800 321-1304 (US only)
E-Mail: communications@karlstorz.com

STORZ
KARL STORZ—ENDOSKOPE

THE DIAMOND STANDARD

KARL STORZ SE & Co. KG
Dr.-Karl-Storz-Straße 34, 78532 Tuttlingen/Germany
Postbox 230, 78503 Tuttlingen/Germany
Phone: +49 7461 708-0
Fax: +49 7461 708-105
E-Mail: info@karlstorz.com

www.karlstorz.com